



SkinMed®

SkinMed® HydroGel HY



Cymedica, spol. s r.o.
info@cymedica.cz
www.cymedica.com

Pod Nádražím 308
Hořovice 268 01
Czech Republic





SkinMed HydroGel HY

Hydrogel 50g

Characteristics:

SkinMed HydroGel HY is a veterinary product in the form of a gel with hyaluronic acid to promote healing of damaged skin and mucous membranes. The gel hydrates the wound, reduces microbial load, prevents secondary infection, creates a moist environment, thus helping the body to develop its own healing process. It helps to stop bleeding, soothes defects, reduces pain and swelling. Hyaluronic acid accelerates the healing process, prevents collagen deposition and thus eliminates scarring.

The consistency of the gel is optimal for application in all positions (it does not flow), it adheres perfectly to the wound site. Application is painless, non-stinging and non-irritating. The gel is also safe to lick or swallow.

The product can be used for all target animal species, except fish and amphibians.

Ingredients:

SkinMedSuper solution (94.96%), sodium hyaluronate (1.44%), hydroxyethylcellulose, preservative, sodium tetraborate

Use:

For topical application on non-healing wounds (ulcers, cracks, fissures, fistulas) and burns, decayed surgical wounds, post-operative and post-traumatic wounds. The treatment has an antimicrobial and regenerative effect. It is mainly used on wounds in the granulation and epithelial phase of healing.

First Aid Instructions:

In contact with the skin: the product is not irritating, only in individuals very sensitive to chlorine a slight irritation may occur, which disappears after a short time.

In case of eye contact: the product is not irritating, in very sensitive individuals wash eyes with running water if irritation occurs.



In case of ingestion: no precautions are necessary in case of ingestion of less than 1 dcl of the product. In case of ingestion of a larger quantity, let the animal drink (if possible) a larger quantity of water (1 dcl) and call a veterinarian.

Instructions for use:

Clean the wound of mechanical debris. Rinse the wound with SkinMed Super rinse solution for better effect. Then apply the gel directly on the defect, so that it follows the edges of the wound in a sufficient layer. The wound can be covered with a non-adherent covering and the gel applied once every 1-2 days. In the case of healing without coverage, we recommend application 2-3 times a day. Combination with other preparations is possible, including injection of antibiotics, non-steroidal antiphlogistic agents and other preparations. Do not use simultaneously with topical ATB (antibiotics) in collagen carriers. The local effects of ATBs may be reduced.

Handling and storage:

Store in original containers out of the reach of children in a cool, dark place for long periods of time; short-term (during use) storage up to a maximum temperature of 30°C. Small quantities may be disposed of by diluting with water and flushing down the drain. The product is a clear gel. Empty containers after rinsing with water can be disposed of with municipal waste.

Caution:

Do not use in case of allergies to any of the substances. Store between 5°C - 30°C

Shelf life:

18 months

Distributor and marketing authorisation holder:

Cymedica spol. sr.o.
Pod Nádražím 308, Hořovice 268 01
Czech Republic
www.cymedica.com