



SkinMed®

SkinMed® Super HydroGel



Cymedica, spol. s r.o.
info@cymedica.cz
www.cymedica.com

Pod Nádražím 308
Hořovice 268 01
Czech Republic





SkinMed Super HydroGel

Hydrogel 30g, 50g, 120g, 150g, 250g, 500g

Characteristics:

SkinMedSuper HydroGel is a hydroactive gel designed for the treatment of minor superficial wounds, contributing to faster healing. The application is painless, non-stinging and non-irritating. SkinMed Super HydroGel hydrates the wound, reduces the microbial load on the wound, reduces the risk of secondary infection, creates a moist environment and lowers the alkaline pH in the wound, helping the body to develop its own healing process. The consistency of the gel is optimal for application in all positions (it does not flow), it adheres perfectly to the site of the wound or skin defect. It ensures a non-traumatic wound dressing. It contributes to a shorter healing time. Non-allergenic, safe also when licked or swallowed.

The product can be used for all target animal species, except fish and amphibians.

Ingredients:

SkinMed Super Solution (98.13%), carbomer, triethanolamine

Use:

SkinMed Super HydroGel is designed for the treatment of all types of minor surface wounds and contributes to faster healing. It is also suitable for use in the complementary care of minor burns, minor decubitus, bruises, skin ulcers and skin irritations. It is suitable for routine mechanical treatment of minor wounds, for the treatment of interstitial spaces and skin folds.

Instructions for use:

Clean the affected area of mechanical debris including hair, fur, feathers, etc. Thoroughly rinse the affected area down to the base with SkinMed Super rinse solution. Then apply the gel from the original packaging directly on the defect, so that it follows the edges of the defect in a sufficiently thick layer. If it is necessary to use a cover, apply a sufficient amount of gel directly onto the defect at each replacement. Apply 3-4 times a day until the wound has healed. The gel is compatible with all wound covering materials. If necessary, combinations with other products as determined by the veterinarian may be used in consultation with the veterinarian.

**First Aid Instructions:**

In contact with the skin: the product is not irritating, only in individuals very sensitive to chlorine a slight irritation may occur, which disappears after a short time. In case of eye contact: the product is not irritating, in very sensitive individuals wash eyes with running water if irritation occurs.

In case of ingestion: no precautions are necessary in case of ingestion of less than 1 dcl of the product. In case of ingestion of larger quantities, let the animal drink (if possible) more water and seek veterinary advice.

Handling and storage:

Keep in original containers out of sight and reach of children in a cool, dark place for long periods of time; short-term storage (during use) is possible up to a maximum temperature of 30°C. Small quantities may be disposed of by diluting with water and flushing down the drain. The product is a clear gel. Empty containers after rinsing with water can be disposed of with municipal waste.

Caution:

This product is not a substitute for veterinary care and medications recommended by a veterinarian. Do not use in case of allergies to any of the substances. For animals only.

Shelf life:

18 months

Distributor and marketing authorisation holder:

Cymedica spol. sr.o.
Pod Nádražím 308, Hořovice 268 01
Czech Republic
www.cymedica.com